

# PROPER Set Loading for the Medley™ Pump Module

## LOADING the Administration Set:

1. Adjust the roller clamp to gravity prime the infusion set. When fully primed, stop flow by moving the roller to the closed position.




2. Hold the set upper fitment above the receptacle at the top of the Medley™ Pump Module.




3. Lower the set upper fitment into the receptacle.




4. Position the Flo-Stop® fitment near the recess in the Medley™ Pump Module. Ensure the tubing is not twisted.

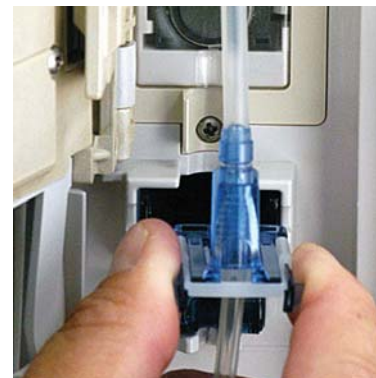

5. Press the Flo-Stop® fitment into the recess in the Medley™ Pump Module.

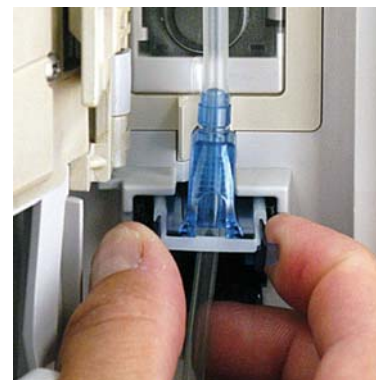

6. Press the tubing into the Air-in-Line Sensor located just below the Flo-Stop® fitment.

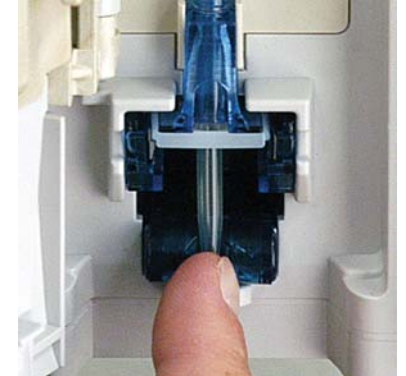

7. Close the door by grasping the Medley™ Pump Module's housing. Ensure that the door is unobstructed.

**WARNING: Do Not Touch the set tubing or upper fitment while closing the door. Failure to follow this instruction may result in infusion rate inaccuracy.**

8. While holding the door closed, gently lower the door latch.

9. Open the roller clamp and begin the infusion.

